



To:

THE DIRECTOR GENERAL OF HEALTH SERVICES IN ALL GOVERNORATES
Commanding Officer, Armed Forces Hospital (Al Khoudh & Salalah)
Director General of Engineering Affairs, MOH
Director General of Royal Hospital
Director General of Khoula Hospital
Director General of Medical Supplies (MOH)
Director General of Pvt. Health Est. Affairs (to kindly arrange distribution to all Pvt. Hospitals)
Hospital Director (Al Nahda Hospital)
Hospital Director (Al Massara Hospital)
The Head of Medical Services in SQU Hospital
The Head of Medical Services in Royal Oman Police
The Head of Medical Services in Ministry of Defence
The Head of Medical Services in The Diwan
The Head of Medical Services in The Sultan's Special Force
The Head of Medical Services in Internal Security Services
The Head of Medical Services in Petroleum Development of Oman
The Head of Medical Services in LNG Oman
ALL PRIVATE PHARMACIES & DRUG STORES

After Compliments,

Please find attached our Circular No 215 dated 23/10/2023 Regarding NCMDR FSN of Olympus Flexible Endoscopes from (mfr: Olympus Medical Systems Corp).

Copy to:

- Director, Office of H.E. The Undersecretary for Health Affairs
- Director of Medical Device Control, DGPA&DC
- Director of Pharmacovigilance & Drug Information Dept, DGPA&DC
- Director of Drug Control Department, DGPA&DC
- Director of Pharmaceutical Licensing Department, DGPA&DC
- Director of Central Quality Control Lab., DGPA&DC
- Supdt. of Central Drug Information





Circular No. 215/ 2023

نتقدم بثقة
Moving Forward
With Confidence



08 -04-1445 H

23 -10-2023

Field Safety Notice of Olympus Flexible Endoscopes from Olympus Medical Systems Corp.

Source	NCMDR- National Center for Medical Devices Reporting- SFDA https://ncmdr.sfda.gov.sa/Secure/CA/CAViewRecall.aspx?caid=4&rid=19729
Product	Olympus Flexible Endoscopes.
Description	Endoscopes.
Manufacturer	Olympus Medical Systems Corp.
Local Agent	Muscat Pharmacy & Stores LLC.
The affected products	Refer to the attachment
Reason	Following repair activities, Olympus discovered that their channel air drying process after water flush of endoscope channels was not validated, and that a small percentage of scopes returned to customers after repair had a wet channel.
Action	1. Inspect each Olympus flexible endoscope returned to you from an Olympus repair facility prior to reprocessing. 2. If you observe any moisture or water, you are requested to send the scope back to the repair center. 3. Until Olympus have implemented the validated process, you will receive a reminder with each returned repaired device. 4. Contact the local agent for remedial action.
comments	Healthcare professionals are encouraged to report any adverse events Suspected to be associated with the above device or any other medical device to Department of Medical Device Control through the E-mail: Med-device@moh.gov.om

Dr. Mohammed Hamdan Al Rubaie
Director General





Attachment: Affected model number list

BF-1T180	BF-1TQ290	BF-H1200	BF-P190	BF-Q180-AC	BF-XP190
BF-1T20D	BF-20D	BF-H190	BF-P260F	BF-Q190	BF-XP260F
BF-1T260	BF-260	BF-H290	BF-P290	BF-Q290	BF-XP290
BF-1T60	BF-3C160	BF-MP160F	BF-P30	BF-TE2	BF-XP60
BF-1TH1100	BF-3C30	BF-MP190F	BF-P40	BF-UC180F	BF-XT160
BF-1TH1200	BF-3C40	BF-MP290F	BF-P60	BF-UC190F	BF-XT190
BF-1TH190	BF-6C260	BF-MP60	BF-PE	BF-UC260FW	
BF-1TQ170	BF-F260	BF-P150	BF-PE2	BF-UC290F	
BF-1TQ180	BF-H1100	BF-P180	BF-Q170	BF-XP160F	

CF-2T160I	CF-H170I	CF-H190I	CF-HQ1100DL	CF-Q160DI	CF-Q180AL
CF-2T160L	CF-H170L	CF-H190L	CF-HQ190I	CF-Q160DL	CF-Q260AL
CF-E3I	CF-H180AI	CF-H260AL	CF-HQ190L	CF-Q160S	CF-Q260DL
CF-E3L	CF-H180AL	CF-H260AZL	CF-HQ290I	CF-Q160ZI	CF-UM20
CF-EZ1500DI	CF-H180DI	CF-H260DL	CF-HQ290L	CF-Q160ZL	CF-XZ1200I
CF-EZ1500DL	CF-H180DL	CF-H290ECI	CF-HQ290ZI	CF-Q165I	CF-XZ1200L
CF-FH260AZI	CF-H185I	CF-H290L	CF-HQ290ZL	CF-Q165L	
CF-FH260AZL	CF-H185L	CF-HQ1100DI	CF-LV1L	CF-Q180AI	

CHF-B290	CHF-BP30	CHF-CB30L	CHF-P60	CHF-V
----------	----------	-----------	---------	-------

CYF-4	CYF-5	CYF-V2	CYF-VH	CYF-VHR	CYF-VHA
CYF-4A	CYF-5A	CYF-VA2			

ENF-T3	ENF-VT	ENF-VT2	ENF-VT3
--------	--------	---------	---------

GF-UC140P-AL5	GF-UCT180	GF-UCT260	GF-UE160-AL5	GF-UE190	GF-UE290
---------------	-----------	-----------	--------------	----------	----------

GIF-1100	GIF-2TQ260M	GIF-H185	GIF-H290Z	GIF-Q180	GIF-XP260N
GIF-1T100	GIF-E3	GIF-H190	GIF-HQ190	GIF-Q260	GIF-XP290N
GIF-1T140	GIF-EZ1500	GIF-H190N	GIF-HQ290	GIF-V70	GIF-XQ260
GIF-1TH190	GIF-FQ260Z	GIF-H260	GIF-LV1	GIF-XP170N	GIF-XTQ160
GIF-1TQ160	GIF-H170	GIF-H290	GIF-N180	GIF-XP180N	GIF-XZ1200
GIF-2T160	GIF-H180	GIF-H290EC	GIF-Q160Z	GIF-XP190N	
GIF-2TH180	GIF-H180J	GIF-H290T	GIF-Q165	GIF-XP260	

HYF-XP

OLYMPUS®

JF-140R	JF-260V
---------	---------

LF-DP	LF-GP	LF-TP	LF-V
-------	-------	-------	------

MAF-DM2	MAF-GM2	MAF-TM2
---------	---------	---------

PCF-H180AI	PCF-H190I	PCF-H290DL	PCF-H290TL	PCF-HQ190L	PCF-Q180AL
PCF-H180AL	PCF-H190L	PCF-H290I	PCF-H290ZI	PCF-PH190I	PCF-Q260AL
PCF-H190DI	PCF-H190TI	PCF-H290L	PCF-H290ZL	PCF-PH190L	
PCF-H190DL	PCF-H190TL	PCF-H290TI	PCF-HQ190I	PCF-Q180AI	

SIF-H190	SIF-H290S	SIF-Q180	SIF-Q260
----------	-----------	----------	----------

TGF-UC180J

TJF-140R	TJF-145	TJF-160VR	TJF-260V	TJF-Q180V	TJF-Q190V
TJF-Q290V					

URF-P6	URF-P7	URF-V	URF-V2R	URF-V3	URF-V3R
URF-P6R	URF-P7R	URF-V2			