



To:

**THE DIRECTOR GENERAL OF HEALTH SERVICES IN ALL GOVERNORATES**

**Commanding Officer, Armed Forces Hospital (Al Khoudh & Salalah)**

**Director General of Engineering Affairs, MOH**

**Director General of Royal Hospital**

**Director General of Khoula Hospital**

**Director General of Medical Supplies (MOH)**

**Director General of Pvt. Health Est. Affairs (to kindly arrange distribution to all Pvt. Hospitals)**

**Hospital Director (Al Nahda Hospital)**

**Hospital Director (Al Massara Hospital)**

**The Head of Medical Services in SQU Hospital**

**The Head of Medical Services in Royal Oman Police**

**The Head of Medical Services in Ministry of Defence**

**The Head of Medical Services in The Diwan**

**The Head of Medical Services in The Sultan's Special Force**

**The Head of Medical Services in Internal Security Services**

**The Head of Medical Services in Petroleum Development of Oman**

**The Head of Medical Services in LNG Oman**

**ALL PRIVATE PHARMACIES & DRUG STORES**

After Compliments,

Please find attached our Circular No 178 dated 24/8/23 Regarding NCMDR  
FSN of Laryngoscopes, flexible, Laryngoscopes, Bronchoscopes from (mfr: Olympus).

Copy to:

- Director, Office of H.E. The Undersecretary for Health Affairs
- Director of Medical Device Control, DGPA&DC
- Director of Pharmacovigilance & Drug Information Dept, DGPA&DC
- Director of Drug Control Department, DGPA&DC
- Director of Pharmaceutical Licensing Department, DGPA&DC
- Director of Central Quality Control Lab., DGPA&DC
- Supdt. of Central Drug Information





Circular No. 178/2023

08 -02-1445 H

24 -08-2023

Field Safety Notice of Laryngoscopes, flexible, Laryngoscopes, Bronchoscopes from Olympus

Source	NCMDR- National Center for Medical Devices Reporting- SFDA <a href="https://ncmdr.sfda.gov.sa/Secure/CA/CaViewRecall.aspx?caid=4&amp;rid=19666">https://ncmdr.sfda.gov.sa/Secure/CA/CaViewRecall.aspx?caid=4&amp;rid=19666</a>
Product	Bronchofiberscope, Bronchovideoscope, Ultrasonic Bronchofibervideoscope, Tracheal Intubation Fiberscope, Tracheal Intubation Videoscope, and Airway Mobilescope.
Description	Laryngoscopes, flexible, Laryngoscopes, Bronchoscopes.
Manufacturer	Olympus.
Local agent	Muscat Pharmacy & Stores LLC.
The affected products	Multiple model numbers (see the attachment). All serial numbers.
Reason	Potential of injury that upon trying to use an Olympus pulmonary endoscope model with an endotracheal tube, the tip may become lodged (entrapped) inside the endotracheal tube connector, due to the scope (bending section) is too large for the endotracheal tube connector.
Action	1. Please refer to the attachment which is providing the additional information related to safe using an endotracheal tube. 2. Contact the local agent for remedial action.
comments	Healthcare professionals are encouraged to report any adverse events Suspected to be associated with the above device or any other medical device to Department of Medical Device Control through the E-mail: <a href="mailto:Med-device@moh.gov.om">Med-device@moh.gov.om</a>

Dr. Mohammed Hamdan Al Rubaie

Director General



PADC  
المديرية العامة للصيدلة والرقابة الدوائية  
Directorate General of Pharmaceutical  
Affairs & Drug Control



ص.ب: 393 مسقط - الرمز البريدي: 100 - هاتف: 22357111 - فاكس: 22358489

P.O. Box: 393 Muscat - Postal Code: 100 - Tel: 22357111 - Fax: 22358489

dgpa\_dc Email: dg-padc@moh.gov.om

## URGENT FIELD SAFETY NOTICE

**Attention:** Endoscopy Examination Room Manager, Operating Room Manager, Risk Management Department

**RE:** Bronchofiberscope, Bronchovideoscope, Ultrasonic Bronchofibervideoscope, Tracheal Intubation Fiberscope, Tracheal Intubation Videoscope, Airway Mobilescope

**Serial numbers:** all serial numbers

Dear Health Care Practitioner,

Olympus has become aware of a matter that requires your attention. This Field Safety Notice pertains to the below-referenced Olympus bronchoscopes/tracheal intubation scopes/airway mobilescopes models. Our records indicate that your facility has purchased one or more of these models.

### BF Series Bronchoscopes

These bronchoscopes are intended for use in endoscopic diagnosis and treatment within the airways and the tracheobronchial tree.

BF-1T150	BF-260	BF-MP160F	BF-P290	BF-TE2	BF-UC180F
BF-1T180	BF-3C160	BF-MP190F	BF-P60	BF-XP160F	BF-UC260FW
BF-1T260	BF-3C40	BF-MP290F	BF-PE2	BF-XP190	BF-UC190F
BF-1T60	BF-6C260	BF-MP60	BF-Q170	BF-XP260F	BF-UC290F
BF-1TQ170	BF-F260	BF-P150	BF-Q180	BF-XP290	—
BF-1TQ180	BF-H190	BF-P180	BF-Q180-AC	BF-XP60	—
BF-1TQ290	BF-H290	BF-P190	BF-Q190	BF-XT160	—
BF-1TH190	BF-N20	BF-P260F	BF-Q290	BF-XT190	—

Note: Product availability is dependent upon country

### LF Series tracheal intubation scopes

These tracheal intubation scopes are intended for airway management which includes endoscopic observation to access airway anatomy, endotracheal/endobronchial intubation and management.

LF-TP	LF-DP	LF-GP	LF-V	LF-P
-------	-------	-------	------	------

Note: Product availability is dependent upon country

### MAF Series airway mobilescopes

These airway mobilescopes are intended for airway management, which includes diagnosis and observation to access airway anatomy, endotracheal/endobronchial intubation and management.

MAF-TM	MAF-TM2	MAF-GM	MAF-GM2	MAF-DM2
--------	---------	--------	---------	---------

Note: Product availability is dependent upon country