

To:

THE DIRECTOR GENERAL OF HEALTH SERVICES IN ALL GOVERNORATES
Commanding Officer, Armed Forces Hospital (Al Khoudh & Salalah)
Director General of Engineering Affairs, MOH
Director General of Royal Hospital
Director General of Khoula Hospital
Director General of Medical Supplies (MOH)
Director General of Pvt. Health Est. Affairs (to kindly arrange distribution to all Pvt. Hospitals)
Hospital Director (Al Nahda Hospital)
Hospital Director (Al Massara Hospital)
The Head of Medical Services in SQU Hospital
The Head of Medical Services in Royal Oman Police
The Head of Medical Services in Ministry of Defence
The Head of Medical Services in The Diwan
The Head of Medical Services in The Sultan's Special Force
The Head of Medical Services in Internal Security Services
The Head of Medical Services in Petroleum Development of Oman
The Head of Medical Services in LNG Oman
ALL PRIVATE PHARMACIES & DRUG STORES


After Compliments,

Please find attached our Circular No 238 dated 28/12/2022 Regarding NCMDR Field Safety Corrective Action of Polaris Multimedia from (mfr: Draeger Medical Systems Inc).

Copy to:

- Director, Office of H.E. The Undersecretary for Health Affairs
- Director of Medical Device Control, DGPA&DC
- Director of Pharmacovigilance & Drug Information Dept, DGPA&DC
- Director of Drug Control Department, DGPA&DC
- Director of Pharmaceutical Licensing Department, DGPA&DC
- Director of Central Quality Control Lab., DGPA&DC
- Supdt. of Central Drug Information



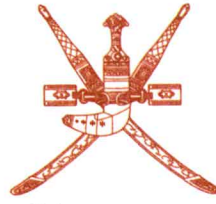

Circular No. 238 / 2022

04 -06-1444 H

28 -12-2022

نقدم بثقة
Moving Forward
with Confidence



Field Safety Corrective Action of Polaris Multimedia from Draeger Medical Systems Inc.

Source	NCMDR- National Center for Medical Devices Reporting- SFDA https://ncmdr.sfda.gov.sa/Secure/CA/CaViewRecall.aspx?caid=6&rid=17357
Product	Polaris Multimedia
Description	Surgical equipment/ Anaesthesia - surgical equipment
Manufacturer	Draeger Medical Systems Inc
Local agent	Waleed Pharmacy & Stores LLC.
The affected products	Bearing D60 MO, part numbers G28994 and G92759.
Reason	The swivel arm of a display dropped from a ceiling tube, where by it was held in place by internal cabling only
Action	1. Please contact Draeger by using the forms in the attached FSN if there is already mechanical wear in the area of the stop pin or stop of the swivel arm. 2. For the affected systems, Draeger will contact the user to ensure the system is repaired. Depending on the condition of the stops, it may be necessary to replace parts of the installation. 3. Contact the local agent for remedial action.
comments	Healthcare professionals are encouraged to report any adverse events Suspected to be associated with the above device or any other medical device to Department of Medical Device Control through the E-mail: Med-device@moh.gov.om

Dr. Mohammed Hamdan Al Rubaie
Director General



PADC
المديرية العامة للصيدلة والرقابة الدوائية
Directorate General of Pharmaceutical
Affairs & Drug Control



ص.ب: ٣٩٣ مسقط - الرمز البريدي: ١٠٠ - هاتف: ٢٢٣٥٧١١١ - فاكس: ٢٢٣٥٨٤٨٩

P.O. Box: 393 Muscat - Postal Code: 100 - Tel: 22357111 - Fax: 22358489

dgpa_dc Email: dg-padcc@moh.gov.om

To our customers of
Dräger Polaris Multimedia: single display
mounting system (D60 MO)


October 2022

Important Safety Notice

Potentially missing securing element

Only the following products are affected:

Dräger Polaris Multimedia: single display mounting system, ceiling
bearing D60 MO, part numbers G28994 and G92759.

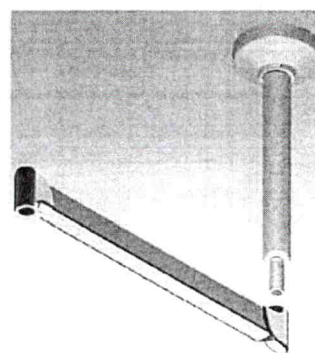

Dear Sir or Madam,

During the course of our global market surveillance activities, we have become aware of a case in which the swivel arm of a display dropped from a ceiling tube, whereby it was held in place by internal cabling only. There was no report of any adverse impact on the health of the patient or user.

The investigation showed that one of three elements which ensure reliable fixation over time between the ceiling tube and the swivel arm, was missing. A missing element leads to mechanical clearance in the solution. If the display mount is moved frequently and with force against the stop, the system may deteriorate internally. The remaining elements may lose their function leading to a disengagement of the swivel arm from the ceiling tube. This may lead to the swivel arm display mount falling which could result in injury.

Actions to be taken:

To verify that in your installation all three elements are mounted correctly, please:



1. Open the cap using a slotted screwdriver. The two halves of the cap can be opened from the bottom.

Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23558 Lübeck, Deutschland
Postanschrift:
23542 Lübeck, Deutschland
Tel. +49 451 882-0
Fax +49 451 882-2080
info@draeger.com
www.draeger.com
UID-Nr. DE135082211


Bankverbindungen:
Commerzbank AG, Lübeck
IBAN: DE95 2304 0022 0014 6795 00
Swift-Code: COBA DE FF 230
Sparkasse zu Lübeck
IBAN: DE15 2305 0101 0001 0711 17
Swift-Code: NOLADE21SPL

Sitz der Gesellschaft: Lübeck
Handelsregister:
Amtsgericht Lübeck HRB 7903 HL
Komplementär: Drägerwerk Verwaltungs AG
Sitz der Gesellschaft: Lübeck
Handelsregister:
Amtsgericht Lübeck HRB 7395 HL


Vorsitzender des Aufsichtsrats der
Drägerwerk AG & Co. KGaA und
Drägerwerk Verwaltungs AG:
Stefan Lauer
Vorstand:
Stefan Dräger (Vorsitzender)
Rainer Klug
Gert-Hartwig Lescow
Dr. Reiner Piske
Anton Schrofner

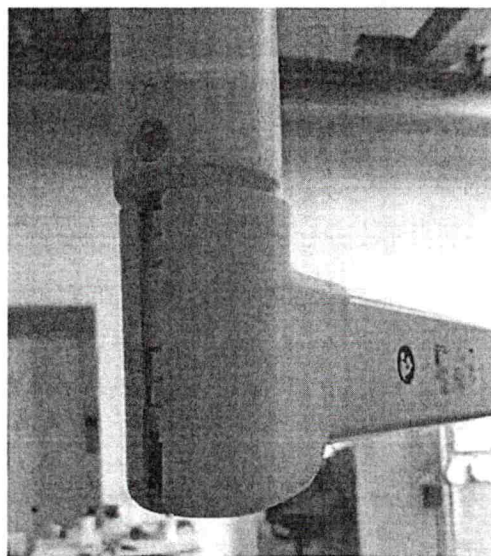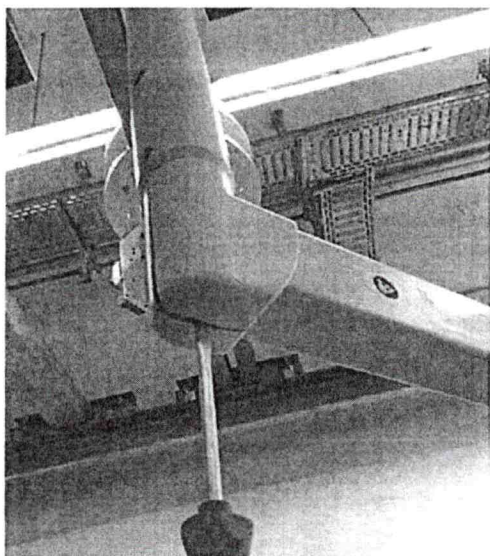

2. Check whether there is a gap between the ceiling tube and the stop at the swivel arm:

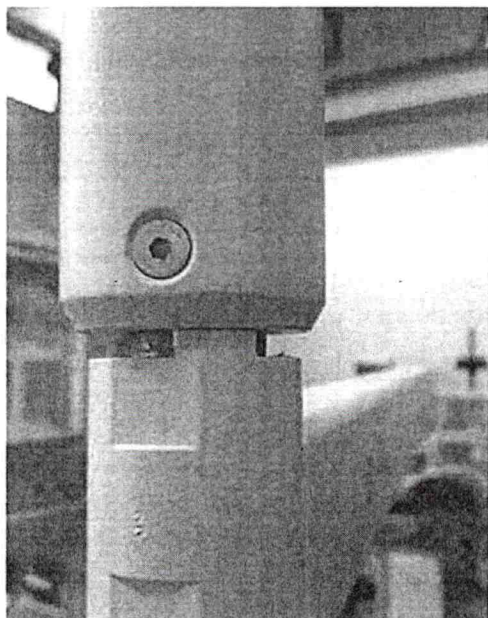

No gap, system is not affected - **OK**

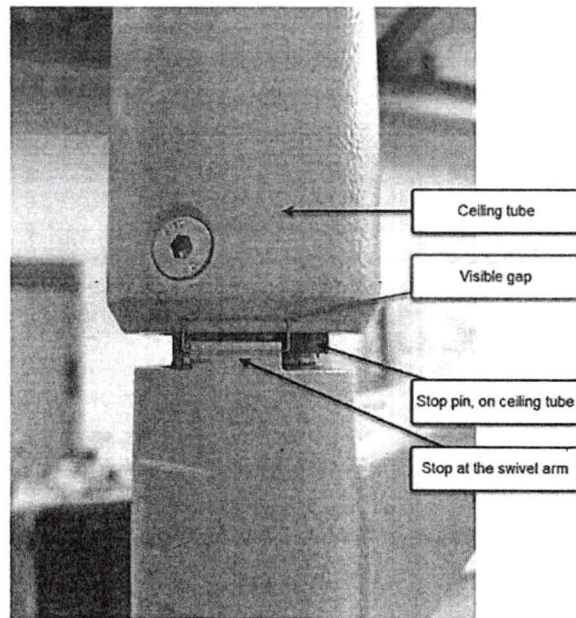

Visible gap, system is affected - **NOT OK**

3. Please contact Draeger by using the attached Reply and Order Card. If your system is affected, we will contact you and ensure the system is repaired free of charge. Depending on the condition of the stops, it may be necessary to replace parts of the installation. Therefore, we ask you to inform us within the Reply and Order Card if there is already mechanical wear in the area of the stop pin or stop of the swivel arm.

Until Dräger has repaired your installation please do not move the device with force against the stops.

If this precaution is followed, the installation can be safely used.

Please ensure that all users of the above-mentioned products and other persons within your organization are made aware of this Important Safety Notice. If you have provided the products to third parties, please forward a copy of this information to them.

Please keep this information at least until the measure has been completed.

The responsible authorities have been notified of this action.

Identification of the affected medical devices:

According to our records, one or more Dräger Polaris Multimedia single display mounting systems manufactured by Drägerwerk AG & Co. KGaA (European Single Registration Number: DE-MF-000005329) that may be affected by this issue are installed in your facility. All serial numbers are potentially affected and are listed in the attachment.

Contact:

If you have any questions, please do not hesitate to contact your local service organization.

Seite 4 / 16

We apologize for any inconvenience caused by this measure.

We thank you for your support.



Director Quality, Regulatory Affairs and Purchasing
Business Unit Workplace Infrastructure
Medical Division
Drägerwerk AG & Co. KGaA

Seite 5 / 16

Attachment:

List of affected medical devices including their Serial Number

Product name	Serial Number
Ceil. Bearing D60 MO	1040710100525
Ceil. Bearing D60 MO	1040710100529
Ceil. Bearing D60 MO	1040710100527
Ceil. Bearing D60 MO	1040710100526
Ceil. Bearing D60 MO	1040710100528
Ceil. Bearing D60 MO	10412939956
Ceil. Bearing D60 MO	1040710100531
Ceil. Bearing D60 MO	1040710100530
Ceil. Bearing D60 MO	1040710100534
Ceil. Bearing D60 MO	1040710100533
Ceil. Bearing D60 MO	10412939958
Ceil. Bearing D60 MO	10412939952
Ceil. Bearing D60 MO	1040710100532
Ceil. Bearing D60 MO	10411939153
Ceil. Bearing D60 MO	10411939152
Ceil. Bearing D60 MO	10411939155
Ceil. Bearing D60 MO	10411939154
Ceil. Bearing D60 MO	10401040447
Ceil. Bearing D60 MO	10401040445
Ceil. Bearing D60 MO	10401040446
Ceil. Bearing D60 MO	10401040448
Ceil. Bearing D60 MO	10401040449
Ceil. Bearing D60 MO	10401040440
Ceil. Bearing D60 MO	10401040441
Ceil. Bearing D60 MO	10401040444
Ceil. Bearing D60 MO	10401040443
Ceil. Bearing D60 MO	10401040442
Ceil. Bearing D60 MO	1041210100637
Ceil. Bearing D60 MO	1041210100641
Ceil. Bearing D60 MO	1041210100636
Ceil. Bearing D60 MO	1041210100633

Ceil. Bearing D60 MO	1041210100639
Ceil. Bearing D60 MO	1041210100638
Ceil. Bearing D60 MO	1041310100691
Ceil. Bearing D60 MO	1041710100725
Ceil. Bearing D60 MO	1041710100723
Ceil. Bearing D60 MO	1041710100726
Ceil. Bearing D60 MO	1041310100690
Ceil. Bearing D60 MO	1041310100688
Ceil. Bearing D60 MO	1041310100694
Ceil. Bearing D60 MO	1041210100697
Ceil. Bearing D60 MO	1041210100634
Ceil. Bearing D60 MO	1041310100692
Ceil. Bearing D60 MO	1041710100724
Ceil. Bearing D60 MO	1041710100727
Ceil. Bearing D60 MO	1041210100635
Ceil. Bearing D60 MO	1041310100693
Ceil. Bearing D60 MO	1041310100696
Ceil. Bearing D60 MO	1042110100791
Ceil. Bearing D60 MO	1042110100784
Ceil. Bearing D60 MO	1042110100787
Ceil. Bearing D60 MO	1042110100786
Ceil. Bearing D60 MO	1042110100783
Ceil. Bearing D60 MO	1041310100689
Ceil. Bearing D60 MO	1042110100785
Ceil. Bearing D60 MO	1042110100789
Ceil. Bearing D60 MO	1042110100792
Ceil. Bearing D60 MO	1042110100790
Ceil. Bearing D60 MO	1041310100695
Ceil. Bearing D60 MO	1042110100788
Ceil. Bearing D60 MO	1043210100988
Ceil. Bearing D60 MO	1043210100993

Ceil. Bearing D60 MO	1043210100989
Ceil. Bearing D60 MO	1041710100730
Ceil. Bearing D60 MO	1041710100728
Ceil. Bearing D60 MO	1041710100729
Ceil. Bearing D60 MO	1043210100992
Ceil. Bearing D60 MO	1041710100732
Ceil. Bearing D60 MO	1041710100731
Ceil. Bearing D60 MO	1043210100996
Ceil. Bearing D60 MO	1043210100991
Ceil. Bearing D60 MO	1043210100995
Ceil. Bearing D60 MO	1043210100987
Ceil. Bearing D60 MO	1044610101407
Ceil. Bearing D60 MO	1044610101404
Ceil. Bearing D60 MO	1044610101402
Ceil. Bearing D60 MO	1044610101409
Ceil. Bearing D60 MO	1044610101408
Ceil. Bearing D60 MO	1044610101410
Ceil. Bearing D60 MO	1044610101406
Ceil. Bearing D60 MO	1044810101462
Ceil. Bearing D60 MO	1043210100990
Ceil. Bearing D60 MO	1043210100994
Ceil. Bearing D60 MO	1044810101456
Ceil. Bearing D60 MO	1041210100640
Ceil. Bearing D60 MO	1044610101403
Ceil. Bearing D60 MO	1044610101405
Ceil. Bearing D60 MO	1044610101401
Ceil. Bearing D60 MO	1044810101455
Ceil. Bearing D60 MO	1044810101459
Ceil. Bearing D60 MO	1044810101457
Ceil. Bearing D60 MO	1044810101463
Ceil. Bearing D60 MO	1044810101454

Ceil. Bearing D60 MO	1044810101458
Ceil. Bearing D60 MO	1044810101460
Ceil. Bearing D60 MO	1045110101500
Ceil. Bearing D60 MO	1045110101501
Ceil. Bearing D60 MO	1045110101503
Ceil. Bearing D60 MO	1045110101502
Ceil. Bearing D60 MO	1040511101605
Ceil. Bearing D60 MO	1045110101499
Ceil. Bearing D60 MO	1040511101607
Ceil. Bearing D60 MO	1040511101604
Ceil. Bearing D60 MO	1040511101602
Ceil. Bearing D60 MO	1040511101601
Ceil. Bearing D60 MO	1040511101606
Ceil. Bearing D60 MO	1045110101497
Ceil. Bearing D60 MO	1045110101498
Ceil. Bearing D60 MO	1040711101609
Ceil. Bearing D60 MO	1040511101600
Ceil. Bearing D60 MO	1045110101495
Ceil. Bearing D60 MO	1040511101603
Ceil. Bearing D60 MO	1040511101599
Ceil. Bearing D60 MO	1040511101608
Ceil. Bearing D60 MO	1045110101496
Ceil. Bearing D60 MO	1040711101612
Ceil. Bearing D60 MO	1040711101615
Ceil. Bearing D60 MO	1040711101616
Ceil. Bearing D60 MO	1040711101614
Ceil. Bearing D60 MO	1040711101618
Ceil. Bearing D60 MO	1040711101617
Ceil. Bearing D60 MO	1040711101611
Ceil. Bearing D60 MO	1042111102022
Ceil. Bearing D60 MO	1042111102020
Ceil. Bearing D60 MO	1042111102021
Ceil. Bearing D60 MO	1043111102232
Ceil. Bearing D60 MO	1043111102231
Ceil. Bearing D60 MO	1043111102229
Ceil. Bearing D60 MO	1043111102236

Ceil. Bearing D60 MO	1043111102230
Ceil. Bearing D60 MO	1043111102237
Ceil. Bearing D60 MO	1043111102238
Ceil. Bearing D60 MO	1042111102016
Ceil. Bearing D60 MO	1042111102019
Ceil. Bearing D60 MO	1042111102018
Ceil. Bearing D60 MO	1042111102017
Ceil. Bearing D60 MO	1042111102024
Ceil. Bearing D60 MO	1042111102023
Ceil. Bearing D60 MO	1042111102025
Ceil. Bearing D60 MO	1040711101613
Ceil. Bearing D60 MO	1043811102459
Ceil. Bearing D60 MO	1043811102456
Ceil. Bearing D60 MO	1043811102455
Ceil. Bearing D60 MO	1043811102462
Ceil. Bearing D60 MO	1043811102460
Ceil. Bearing D60 MO	1044311102584
Ceil. Bearing D60 MO	1043811102457
Ceil. Bearing D60 MO	1043111102233
Ceil. Bearing D60 MO	1040711101625
Ceil. Bearing D60 MO	1043111102234
Ceil. Bearing D60 MO	1044311102582
Ceil. Bearing D60 MO	1043811102458
Ceil. Bearing D60 MO	1043111102235
Ceil. Bearing D60 MO	1043811102454
Ceil. Bearing D60 MO	1043811102453
Ceil. Bearing D60 MO	1043811102461
Ceil. Bearing D60 MO	1044311102583
Ceil. Bearing D60 MO	1044311102587
Ceil. Bearing D60 MO	1044311102585
Ceil. Bearing D60 MO	1044311102581
Ceil. Bearing D60 MO	1044611102591
Ceil. Bearing D60 MO	1044611102599
Ceil. Bearing D60 MO	1044311102580
Ceil. Bearing D60 MO	1044311102586
Ceil. Bearing D60 MO	1044311102589

Ceil. Bearing D60 MO	1044311102588
Ceil. Bearing D60 MO	1044611102595
Ceil. Bearing D60 MO	1044611102598
Ceil. Bearing D60 MO	1044811102948
Ceil. Bearing D60 MO	1044811102949
Ceil. Bearing D60 MO	1044611102594
Ceil. Bearing D60 MO	1044611102597
Ceil. Bearing D60 MO	1044611102600
Ceil. Bearing D60 MO	1044611102593
Ceil. Bearing D60 MO	1044611102592
Ceil. Bearing D60 MO	1045011103011
Ceil. Bearing D60 MO	1045011103010
Ceil. Bearing D60 MO	1044811102956
Ceil. Bearing D60 MO	1044811102950
Ceil. Bearing D60 MO	1044811102954
Ceil. Bearing D60 MO	1044811102952
Ceil. Bearing D60 MO	1044611102596
Ceil. Bearing D60 MO	1044811102951
Ceil. Bearing D60 MO	1044811102953
Ceil. Bearing D60 MO	1044811102955
Ceil. Bearing D60 MO	1044811102957
Ceil. Bearing D60 MO	1045011103018
Ceil. Bearing D60 MO	1045011103016
Ceil. Bearing D60 MO	1045011103015
Ceil. Bearing D60 MO	1040412103441
Ceil. Bearing D60 MO	1045011103014
Ceil. Bearing D60 MO	1040412103440
Ceil. Bearing D60 MO	1040412103438
Ceil. Bearing D60 MO	1040412103439
Ceil. Bearing D60 MO	1040412103442
Ceil. Bearing D60 MO	1045011103013
Ceil. Bearing D60 MO	1045011103012
Ceil. Bearing D60 MO	1045011103017
Ceil. Bearing D60 MO	1040412103446
Ceil. Bearing D60 MO	1040412103444
Ceil. Bearing D60 MO	1041012104151

Ceil. Bearing D60 MO	1041012104150
Ceil. Bearing D60 MO	1041012104145
Ceil. Bearing D60 MO	1041012104144
Ceil. Bearing D60 MO	2812H000046098
Ceil. Bearing D60 MO	1041012104149
Ceil. Bearing D60 MO	1041012104143
Ceil. Bearing D60 MO	2812H0000460100
Ceil. Bearing D60 MO	2812H000046096
Ceil. Bearing D60 MO	4112H000064689
Ceil. Bearing D60 MO	4112H000064695
Ceil. Bearing D60 MO	1041012104147
Ceil. Bearing D60 MO	2812H000046103
Ceil. Bearing D60 MO	2812H000046097
Ceil. Bearing D60 MO	2812H000046086
Ceil. Bearing D60 MO	4612H000071982
Ceil. Bearing D60 MO	4612H000071981
Ceil. Bearing D60 MO	4612H000071989
Ceil. Bearing D60 MO	6412H000071984
Ceil. Bearing D60 MO	2812H000046088
Ceil. Bearing D60 MO	2812H000046090
Ceil. Bearing D60 MO	4612H000071980
Ceil. Bearing D60 MO	4612H000071983
Ceil. Bearing D60 MO	4612H000071986
Ceil. Bearing D60 MO	2812H000046091
Ceil. Bearing D60 MO	4612H000071988
Ceil. Bearing D60 MO	4612H000071987
Ceil. Bearing D60 MO	4612H000071985
Ceil. Bearing D60 MO	4912H000079423
Ceil. Bearing D60 MO	1413H000110246
Ceil. Bearing D60 MO	1413H000110242
Ceil. Bearing D60 MO	1413H000110245
Ceil. Bearing D60 MO	1413H000110243
Ceil. Bearing D60 MO	1413H000110241
Ceil. Bearing D60 MO	4912H000079425
Ceil. Bearing D60 MO	4912H000079424
Ceil. Bearing D60 MO	4912H000079427

Ceil. Bearing D60 MO	4912H000079429
Ceil. Bearing D60 MO	4912H000079431
Ceil. Bearing D60 MO	4912H000079426
Ceil. Bearing D60 MO	4912H000079430
Ceil. Bearing D60 MO	4912H000079428
Ceil. Bearing D60 MO	4912H000079432
Ceil. Bearing D60 MO	2812H000046092
Ceil. Bearing D60 MO	2812H000046094
Ceil. Bearing D60 MO	2812H000046093
Ceil. Bearing D60 MO	2812H000046095
Ceil. Bearing D60 MO	1413H000110237
Ceil. Bearing D60 MO	1413H000110239
Ceil. Bearing D60 MO	1413H000110240
Ceil. Bearing D60 MO	1613H000114677
Ceil. Bearing D60 MO	1413H000110238
Ceil. Bearing D60 MO	1413H000110244
Ceil. Bearing D60 MO	1613H000114681
Ceil. Bearing D60 MO	1613H000114678
Ceil. Bearing D60 MO	2812H000046084
Ceil. Bearing D60 MO	2812H000046089
Ceil. Bearing D60 MO	1613H000114679
Ceil. Bearing D60 MO	1613H000114676
Ceil. Bearing D60 MO	1613H000114694
Ceil. Bearing D60 MO	1613H000114675
Ceil. Bearing D60 MO	1613H000114682
Ceil. Bearing D60 MO	1613H000114686
Ceil. Bearing D60 MO	1613H000114692
Ceil. Bearing D60 MO	1613H000114693
Ceil. Bearing D60 MO	1613H000114680
Ceil. Bearing D60 MO	3213H000144090
Ceil. Bearing D60 MO	2113H000122785
Ceil. Bearing D60 MO	2113H000122783
Ceil. Bearing D60 MO	2113H000122784
Ceil. Bearing D60 MO	2113H000122786
Ceil. Bearing D60 MO	2812H000046087
Ceil. Bearing D60 MO	2113H000122787

Ceil. Bearing D60 MO	2812H000046085
Ceil. Bearing D60 MO	2812H000046102
Ceil. Bearing D60 MO	2113H000122790
Ceil. Bearing D60 MO	2113H000122791
Ceil. Bearing D60 MO	2113H000122792
Ceil. Bearing D60 MO	3213H000144087
Ceil. Bearing D60 MO	3213H000144085
Ceil. Bearing D60 MO	3213H000144094
Ceil. Bearing D60 MO	3213H000144088
Ceil. Bearing D60 MO	3413H000146378
Ceil. Bearing D60 MO	3413H000146377
Ceil. Bearing D60 MO	3213H000144089
Ceil. Bearing D60 MO	3213H000144086
Ceil. Bearing D60 MO	3213H000144093
Ceil. Bearing D60 MO	1613H000114691
Ceil. Bearing D60 MO	1613H000114685
Ceil. Bearing D60 MO	1613H000114687
Ceil. Bearing D60 MO	1613H000114688
Ceil. Bearing D60 MO	1613H000114683
Ceil. Bearing D60 MO	1613H000114690
Ceil. Bearing D60 MO	1613H000114689
Ceil. Bearing D60 MO	1613H000114684
Ceil. Bearing D60 MO	2113H000122788
Ceil. Bearing D60 MO	3213H000144092
Ceil. Bearing D60 MO	3213H000144091
Ceil. Bearing D60 MO	2113H000122789
Ceil. Bearing D60 MO	3413H000146379
Ceil. Bearing D60 MO	3413H000146382
Ceil. Bearing D60 MO	3413H000146383
Ceil. Bearing D60 MO	3413H000146381
Ceil. Bearing D60 MO	3413H000146385
Ceil. Bearing D60 MO	3613H000150344
Ceil. Bearing D60 MO	3613H000150342
Ceil. Bearing D60 MO	3613H000150345
Ceil. Bearing D60 MO	3613H000150346
Ceil. Bearing D60 MO	3613H000150343

Ceil. Bearing D60 MO	3613H000150351
Ceil. Bearing D60 MO	3413H000146380
Ceil. Bearing D60 MO	3413H000146384
Ceil. Bearing D60 MO	3713H000152921
Ceil. Bearing D60 MO	3613H000150347
Ceil. Bearing D60 MO	3713H000152919
Ceil. Bearing D60 MO	3713H000152917
Ceil. Bearing D60 MO	3713H000152918
Ceil. Bearing D60 MO	3713H000152915
Ceil. Bearing D60 MO	3713H000152916
Ceil. Bearing D60 MO	3613H000150348
Ceil. Bearing D60 MO	3713H000152914
Ceil. Bearing D60 MO	3713H000152923
Ceil. Bearing D60 MO	4913H000179069
Ceil. Bearing D60 MO	4913H000179068
Ceil. Bearing D60 MO	3713H000152920
Ceil. Bearing D60 MO	4913H000179072
Ceil. Bearing D60 MO	4913H000179067
Ceil. Bearing D60 MO	4913H000179064
Ceil. Bearing D60 MO	4913H000179054
Ceil. Bearing D60 MO	4913H000179055
Ceil. Bearing D60 MO	4913H000179053
Ceil. Bearing D60 MO	4913H000179066
Ceil. Bearing D60 MO	4913H000179063
Ceil. Bearing D60 MO	4913H000179065
Ceil. Bearing D60 MO	3413H000146386
Ceil. Bearing D60 MO	3613H000150350
Ceil. Bearing D60 MO	3613H000150349
Ceil. Bearing D60 MO	4913H000179062
Ceil. Bearing D60 MO	0214H000190722
Ceil. Bearing D60 MO	4913H000179061
Ceil. Bearing D60 MO	4913H000179071
Ceil. Bearing D60 MO	4913H000179057
Ceil. Bearing D60 MO	3713H000152922
Ceil. Bearing D60 MO	4913H000179056
Ceil. Bearing D60 MO	4913H000179058

Ceil. Bearing D60 MO	0614H000199543
Ceil. Bearing D60 MO	0614H000199539
Ceil. Bearing D60 MO	0214H000190717
Ceil. Bearing D60 MO	0214H000190720
Ceil. Bearing D60 MO	0614H000199535
Ceil. Bearing D60 MO	0614H000199536
Ceil. Bearing D60 MO	0214H000190721
Ceil. Bearing D60 MO	4913H000179060
Ceil. Bearing D60 MO	0214H000190725
Ceil. Bearing D60 MO	0214H000190723
Ceil. Bearing D60 MO	0214H000190724
Ceil. Bearing D60 MO	0214H000190726
Ceil. Bearing D60 MO	0214H000190718
Ceil. Bearing D60 MO	0214H000190719
Ceil. Bearing D60 MO	4913H000179070
Ceil. Bearing D60 MO	4913H000179059
Ceil. Bearing D60 MO	0614H000199540
Ceil. Bearing D60 MO	0614H000199542
Ceil. Bearing D60 MO	0814H000203045
Ceil. Bearing D60 MO	0814H000203047
Ceil. Bearing D60 MO	0814H000203040
Ceil. Bearing D60 MO	0614H000199541
Ceil. Bearing D60 MO	0814H000203044
Ceil. Bearing D60 MO	0614H000199538
Ceil. Bearing D60 MO	0814H000203048
Ceil. Bearing D60 MO	0614H000199537
Ceil. Bearing D60 MO	0614H000199544
Ceil. Bearing D60 MO	1114H000213151
Ceil. Bearing D60 MO	0814H000203043
Ceil. Bearing D60 MO	1114H000213149
Ceil. Bearing D60 MO	1114H000213147
Ceil. Bearing D60 MO	1114H000213150
Ceil. Bearing D60 MO	1114H000213152
Ceil. Bearing D60 MO	0814H000203046
Ceil. Bearing D60 MO	1114H000213145
Ceil. Bearing D60 MO	0814H000203041

Ceil. Bearing D60 MO	1114H000213146
Ceil. Bearing D60 MO	0814H000203039
Ceil. Bearing D60 MO	0814H000203042
Ceil. Bearing D60 MO	1114H000213144
Ceil. Bearing D60 MO	1114H000213148
Ceil. Bearing D60 MO	2614H000245699
Ceil. Bearing D60 MO	3114H000256197
Ceil. Bearing D60 MO	2614H000245702
Ceil. Bearing D60 MO	2614H000245706
Ceil. Bearing D60 MO	1114H000213143
Ceil. Bearing D60 MO	2614H000245705
Ceil. Bearing D60 MO	2614H000245704
Ceil. Bearing D60 MO	3114H000256202
Ceil. Bearing D60 MO	2614H000245703
Ceil. Bearing D60 MO	2614H000245707
Ceil. Bearing D60 MO	2614H000245708
Ceil. Bearing D60 MO	3114H000256205
Ceil. Bearing D60 MO	3114H000256206
Ceil. Bearing D60 MO	3114H000256204
Ceil. Bearing D60 MO	3114H000256203
Ceil. Bearing D60 MO	0315H000322489
Ceil. Bearing D60 MO	0315H000322490
Ceil. Bearing D60 MO	0315H000322492
Ceil. Bearing D60 MO	0315H000322491
Ceil. Bearing D60 MO	4314H000288030
Ceil. Bearing D60 MO	4314H000288028
Ceil. Bearing D60 MO	4314H000288033
Ceil. Bearing D60 MO	4314H000288032
Ceil. Bearing D60 MO	4314H000288031
Ceil. Bearing D60 MO	4314H000288035
Ceil. Bearing D60 MO	4314H000288034
Ceil. Bearing D60 MO	4314H000288037
Ceil. Bearing D60 MO	4314H000288036
Ceil. Bearing D60 MO	2615H000424495
Ceil. Bearing D60 MO	2615H000424493
Ceil. Bearing D60 MO	2615H000424494

Ceil. Bearing D60 MO	2716H000637653
Ceil. Bearing D60 MO	2716H000637648
Ceil. Bearing D60 MO	2716H000637650
Ceil. Bearing D60 MO	2716H000637649
Ceil. Bearing D60 MO	3416H000674566
Ceil. Bearing D60 MO	3416H000674562
Ceil. Bearing D60 MO	3416H000674564
Ceil. Bearing D60 MO	2716H000637646
Ceil. Bearing D60 MO	3416H000674561
Ceil. Bearing D60 MO	3416H000674563
Ceil. Bearing D60 MO	2716H000637656
Ceil. Bearing D60 MO	2716H000637645
Ceil. Bearing D60 MO	2716H000637647
Ceil. Bearing D60 MO	3416H000674550
Ceil. Bearing D60 MO	3416H000674536
Ceil. Bearing D60 MO	3416H000674553
Ceil. Bearing D60 MO	3416H000674539
Ceil. Bearing D60 MO	3416H000674557
Ceil. Bearing D60 MO	3416H000674549
Ceil. Bearing D60 MO	3416H000674538
Ceil. Bearing D60 MO	3416H000674555
Ceil. Bearing D60 MO	3416H000674551
Ceil. Bearing D60 MO	3416H000674540
Ceil. Bearing D60 MO	3416H000674552
Ceil. Bearing D60 MO	3416H000674558
Ceil. Bearing D60 MO	3416H000674537
Ceil. Bearing D60 MO	3416H000674554
Ceil. Bearing D60 MO	3416H000674541
Ceil. Bearing D60 MO	3416H000674556
Ceil. Bearing D60 MO	3416H000674565
Ceil. Bearing D60 MO	3416H000674543
Ceil. Bearing D60 MO	3416H000674542
Ceil. Bearing D60 MO	3416H000674547
Ceil. Bearing D60 MO	3416H000674545
Ceil. Bearing D60 MO	3416H000674548
Ceil. Bearing D60 MO	3716H000691575

Ceil. Bearing D60 MO	3716H000691572
Ceil. Bearing D60 MO	3716H000691576
Ceil. Bearing D60 MO	3716H000691580
Ceil. Bearing D60 MO	3716H000691579
Ceil. Bearing D60 MO	3716H000691581
Ceil. Bearing D60 MO	3716H000691582
Ceil. Bearing D60 MO	3716H000691583
Ceil. Bearing D60 MO	4216H000718310
Ceil. Bearing D60 MO	3716H000691578
Ceil. Bearing D60 MO	4216H000718308
Ceil. Bearing D60 MO	4216H000718311
Ceil. Bearing D60 MO	4216H000718312
Ceil. Bearing D60 MO	4216H000718309
Ceil. Bearing D60 MO	4216H000718295
Ceil. Bearing D60 MO	4216H000718307
Ceil. Bearing D60 MO	4216H000718305
Ceil. Bearing D60 MO	4216H000718294
Ceil. Bearing D60 MO	4216H000718299
Ceil. Bearing D60 MO	4216H000718293
Ceil. Bearing D60 MO	4216H000718298
Ceil. Bearing D60 MO	4216H000718297
Ceil. Bearing D60 MO	4216H000718296
Ceil. Bearing D60 MO	0717H000845265
Ceil. Bearing D60 MO	3316H000673676
Ceil. Bearing D60 MO	3316H000673687
Ceil. Bearing D60 MO	0917H000854213
Ceil. Bearing D60 MO	3316H000673685
Ceil. Bearing D60 MO	0717H000845263
Ceil. Bearing D60 MO	3316H000673683
Ceil. Bearing D60 MO	0717H000845266
Ceil. Bearing D60 MO	3316H000673680
Ceil. Bearing D60 MO	0717H000845256
Ceil. Bearing D60 MO	3316H000673684
Ceil. Bearing D60 MO	3316H000673679
Ceil. Bearing D60 MO	0717H000845257
Ceil. Bearing D60 MO	0717H000845258

Ceil. Bearing D60 MO	0717H000845261
Ceil. Bearing D60 MO	0717H000845259
Ceil. Bearing D60 MO	0717H000845253
Ceil. Bearing D60 MO	0717H000845254
Ceil. Bearing D60 MO	0717H000845252
Ceil. Bearing D60 MO	3416H000674546
Ceil. Bearing D60 MO	3416H000674559
Ceil. Bearing D60 MO	3416H000674560
Ceil. Bearing D60 MO	2217H000929378
Ceil. Bearing D60 MO	2217H000929368
Ceil. Bearing D60 MO	2217H000929377
Ceil. Bearing D60 MO	0717H000845255
Ceil. Bearing D60 MO	0717H000845260
Ceil. Bearing D60 MO	2217H000929379
Ceil. Bearing D60 MO	2217H000929370
Ceil. Bearing D60 MO	2217H000929374
Ceil. Bearing D60 MO	2217H000929375
Ceil. Bearing D60 MO	2217H000929373
Ceil. Bearing D60 MO	2217H000929371
Ceil. Bearing D60 MO	2217H000929376
Ceil. Bearing D60 MO	3217H001001047
Ceil. Bearing D60 MO	10412939951
Ceil. Bearing D60 MO	10412939954
Ceil. Bearing D60 MO	3217H001001048
Ceil. Bearing D60 MO	3217H001001046
Ceil. Bearing D60 MO	3217H001001049
Ceil. Bearing D60 MO	3217H001001056
Ceil. Bearing D60 MO	3217H001001057
Ceil. Bearing D60 MO	3217H001001059
Ceil. Bearing D60 MO	3217H001001051
Ceil. Bearing D60 MO	3217H001001055
Ceil. Bearing D60 MO	3217H001001050
Ceil. Bearing D60 MO	3217H001001058
Ceil. Bearing D60 MO	3217H001001060
Ceil. Bearing D60 MO	3217H001001062
Ceil. Bearing D60 MO	3217H001001061

Ceil. Bearing D60 MO	4717H001109012
Ceil. Bearing D60 MO	4717H001109011
Ceil. Bearing D60 MO	3217H001001065
Ceil. Bearing D60 MO	4717H001109013
Ceil. Bearing D60 MO	4717H001109009
Ceil. Bearing D60 MO	3217H001001053
Ceil. Bearing D60 MO	1118H001217363
Ceil. Bearing D60 MO	3217H001001052
Ceil. Bearing D60 MO	2716H000637660
Ceil. Bearing D60 MO	4717H001109014
Ceil. Bearing D60 MO	1118H001217360
Ceil. Bearing D60 MO	4717H001109006
Ceil. Bearing D60 MO	4717H001109005
Ceil. Bearing D60 MO	3217H001001054
Ceil. Bearing D60 MO	3217H001001063
Ceil. Bearing D60 MO	4717H001109007
Ceil. Bearing D60 MO	2716H000637655
Ceil. Bearing D60 MO	1118H001217358
Ceil. Bearing D60 MO	2716H000637661
Ceil. Bearing D60 MO	4717H001109008
Ceil. Bearing D60 MO	2716H000637658
Ceil. Bearing D60 MO	4717H001109010
Ceil. Bearing D60 MO	3217H001001064
Ceil. Bearing D60 MO	1118H001217352
Ceil. Bearing D60 MO	1118H001217353
Ceil. Bearing D60 MO	1118H001217355
Ceil. Bearing D60 MO	1118H001217369
Ceil. Bearing D60 MO	1118H001217372
Ceil. Bearing D60 MO	1118H001217370
Ceil. Bearing D60 MO	1118H001217371
Ceil. Bearing D60 MO	1118H001217375
Ceil. Bearing D60 MO	1118H001217373
Ceil. Bearing D60 MO	1118H001217374
Ceil. Bearing D60 MO	1118H001217351
Ceil. Bearing D60 MO	1118H001217359
Ceil. Bearing D60 MO	1118H001217365

Ceil. Bearing D60 MO	5019H001935498
Ceil. Bearing D60 MO	5019H001935497
Ceil. Bearing D60 MO	5019H001935500
Ceil. Bearing D60 MO	5019H001935499
Ceil. Bearing D60 MO	5019H001935507
Ceil. Bearing D60 MO	5019H001935490
Ceil. Bearing D60 MO	4719H001908593
Ceil. Bearing D60 MO	4719H001908589
Ceil. Bearing D60 MO	4719H001908590
Ceil. Bearing D60 MO	4719H001908591
Ceil. Bearing D60 MO	4419H001884621
Ceil. Bearing D60 MO	4419H001884622
Ceil. Bearing D60 MO	4719H001908580
Ceil. Bearing D60 MO	4719H001908581
Ceil. Bearing D60 MO	4719H001908577
Ceil. Bearing D60 MO	4419H001880711
Ceil. Bearing D60 MO	4719H001908592
Ceil. Bearing D60 MO	4719H001908583
Ceil. Bearing D60 MO	4719H001908585
Ceil. Bearing D60 MO	4719H001908588
Ceil. Bearing D60 MO	4719H001908587
Ceil. Bearing D60 MO	4719H001908586
Ceil. Bearing D60 MO	4719H001908584
Ceil. Bearing D60 MO	4719H001908579
Ceil. Bearing D60 MO	4719H001908578
Ceil. Bearing D60 MO	4719H001908582
Ceil. Bearing D60 MO	0720H002015762
Ceil. Bearing D60 MO	0720H002016431
Ceil. Bearing D60 MO	0720H002015761
Ceil. Bearing D60 MO	0720H002016426
Ceil. Bearing D60 MO	2217H000929369
Ceil. Bearing D60 MO	2217H000929372
Ceil. Bearing D60 MO	0720H002016428
Ceil. Bearing D60 MO	0720H002016430
Ceil. Bearing D60 MO	0720H002016429
Ceil. Bearing D60 MO	4418H001498062

Seite 16 / 16

Ceil. Bearing D60 MO for CSU	4719H001908539
Ceil. Bearing D60 MO for CSU	4719H001908535
Ceil. Bearing D60 MO for CSU	4719H001908540
Ceil. Bearing D60 MO for CSU	4719H001908543
Ceil. Bearing D60 MO for CSU	4719H001908534
Ceil. Bearing D60 MO for CSU	4719H001908545
Ceil. Bearing D60 MO for CSU	4719H001908542
Ceil. Bearing D60 MO for CSU	4719H001908538
Ceil. Bearing D60 MO for CSU	4719H001908544
Ceil. Bearing D60 MO for CSU	5119H001947132
Ceil. Bearing D60 MO for CSU	5019H001942411
Ceil. Bearing D60 MO for CSU	5119H001947131
Ceil. Bearing D60 MO for CSU	5019H001942412

Ceil. Bearing D60 MO for CSU	5019H001942409
Ceil. Bearing D60 MO for CSU	5019H001942410
Ceil. Bearing D60 MO for CSU	5019H001942407
Ceil. Bearing D60 MO for CSU	5019H001942408
Ceil. Bearing D60 MO for CSU	5119H001954602
Ceil. Bearing D60 MO for CSU	5119H001954605
Ceil. Bearing D60 MO for CSU	5119H001954604
Ceil. Bearing D60 MO for CSU	5119H001954603
Ceil. Bearing D60 MO for CSU	5119H001954606
Ceil. Bearing D60 MO for CSU	5119H001954607
Ceil. Bearing D60 MO for CSU	2164620
Ceil. Bearing D60 MO for CSU	2164618
Ceil. Bearing D60 MO for CSU	2164619

Ceil. Bearing D60 MO for CSU	1947133
Ceil. Bearing D60 MO for CSU	1947136
Ceil. Bearing D60 MO for CSU	2838693
Ceil. Bearing D60 MO for CSU	1947135
Ceil. Bearing D60 MO for CSU	1947134
Ceil. Bearing D60 MO for CSU	2164612
Ceil. Bearing D60 MO for CSU	2164611
Ceil. Bearing D60 MO for CSU	2164615
Ceil. Bearing D60 MO for CSU	2164616
Ceil. Bearing D60 MO for CSU	2164617
Ceil. Bearing D60 MO for CSU	2838696
Ceil. Bearing D60 MO for CSU	2838697